

TIME SERIES



INSTACHK HIV 1+2

One Step Anti-HIV (1&2) Tri-Line Test



Features

- Rapid, Visual, Qualitative detection of antibodies of all isotopes (IgG, IgM, IgA) specific to HIV-1 including subtype O and HIV-2 simultaneously in human whole blood, serum or plasma
- Complete Kit. No external pretreatment
- Pack Size 40 Tests; Storage 2-30° C
- Excellent result correlation with EIA and Western Blot
- Sensitivity 100%; Specificity 100%
- P24 antigen incorporated (in addition to gp41, gp120 & gp36)
- No equipment required

INSTACHK HIV 1+2


One Step Anti-HIV (1&2) Tri-Line Test

Test Principle

Colloidal Gold Enhanced, Rapid Immunochromatographic Assay

Test Procedure

1. Dispense 1 drop (30 μ L) of specimen using the plastic dropper into the sample well.
2. Add one drop of sample diluent into the same well.
3. Interpret test results in 15 minutes.

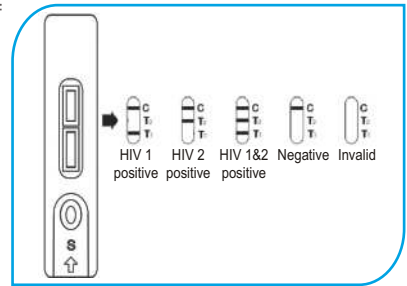

Interpretation

1. **Positive:** Presence of coloured control line and at least one test line on the membrane. T1 test line indicates an HIV-1 positive result. T2 test line indicates an HIV-2 positive result. The appearing of both T1 and T2 test lines indicate HIV 1 & 2 positive result.
2. **Negative:** Only presence of coloured control line.
3. **Invalid:** Absence of coloured control band (Repeat with a new test device).

Imported and Marketed exclusively by :

ERBA diagnostics Mannheim GmbH

Mallaustrasse 69-73, 68219 Mannheim / Germany
Telephone: +49 621 8799770 Fax: +49 621 8799688
Website: www.erbamannheim.com
Email : sales@erbamannheim.com

Transasia Bio-Medicals Ltd.

Transasia House, 8 Chandivali Studio Road, Andheri (E), Mumbai - 400 072
Tel.: (022) 40309000, Fax : (022) 2857 3030
Email : transasia@transasia.co.in