


INSTACHK HEPATITIS B

One Step HBsAg Test



HBV antigens and antibodies in the blood

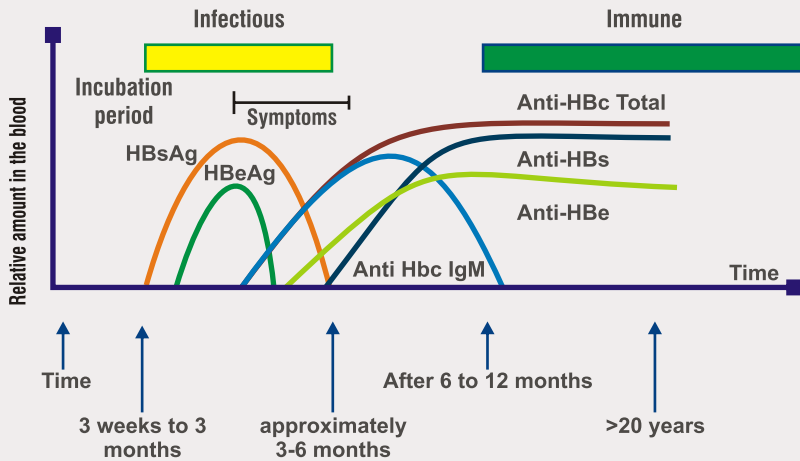

Hepatitis B viral antigens and antibodies detectable in the blood

- ✦ Rapid, Visual, Qualitative detection of Hepatitis B surface antigen (HBsAg) in human serum, plasma or whole blood
- ✦ Complete Kit. No external pretreatment
- ✦ Sensitivity: 1 ng/mL (including both ad and ay subtypes)
- ✦ Pack Size 40 Tests
- ✦ Results can be read within 15 minutes
- ✦ Excellent result correlation with EIA & RIA
- ✦ Storage 2-30° C
- ✦ No equipment required

INSTACHK HEPATITIS B

One Step HBsAg Test

40 Tests

INSTACHK HEPATITIS B


Product Code : FRERAIT10006

★ Test Principle

Colloidal Gold Enhanced, Rapid Immunochromatographic Assay

★ Test Procedure

1. Dispense 3 drops (100 μ L) of specimen using the dropper into the circular sample well.
2. Interpret test results in 15 minutes.

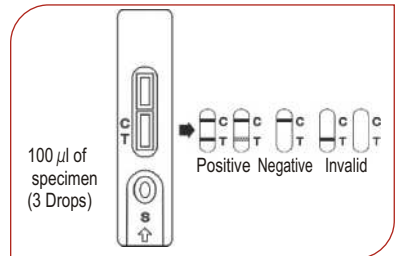

★ Interpretation

1. Positive : Presence of coloured test and control band.
2. Negative : Only presence of coloured control band.
3. Invalid : Absence of coloured control band (Repeat with a new test device).

★ Performance Characteristics

Comparison with RIA : Sensitivity = 100%
Specificity = 99.43%

Imported and Marketed exclusively by :

ERBA diagnostics Mannheim GmbH

Mallastrasse 69-73, 68219 Mannheim / Germany
Telephone: +49 621 8799770 Fax: +49 621 8799688
Website: www.erbamannheim.com
Email : sales@erbamannheim.com

Transasia Bio-Medicals Ltd.

Transasia House, 8 Chandivali Studio Road, Andheri (E), Mumbai - 400 072
Tel.: (022) 40309000, Fax : (022) 2857 3030
Email : transasia@transasia.co.in