


INSTACHK TP

TIME SERIES



One Step Anti-TP (*Treponema pallidum*/Syphilis) Test

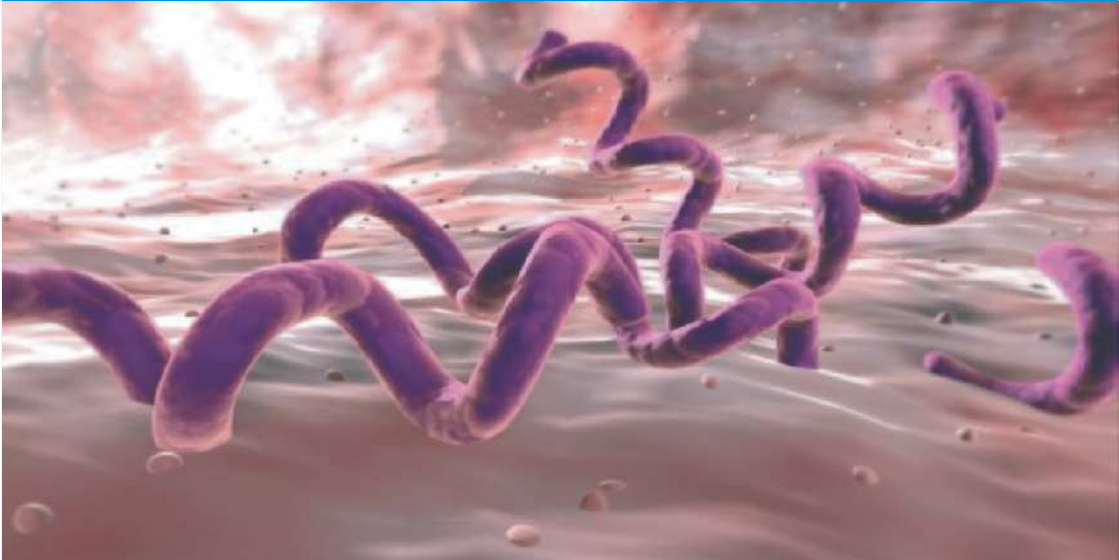

Features :

Treponemal specific test

Rapid, Visual, Qualitative detection of antibodies to *Treponema pallidum* antigen in human whole blood, serum or plasma

Complete Kit. No external pretreatment

Pack Size 40 Tests

Sandwich principle using recombinant immobilized TP antigen (TP Ag 2) on the test band region and another TP antigen (TP Ag 1) coupled with colloidal gold particles dried on a conjugate pad

Excellent correlation of results with EIA

Storage 2-30° C

No equipment required

INSTACHK TP

One Step Anti-TP (*Treponema pallidum*/Syphilis) Test

40 Tests

Test Principle


Double Antigens “Sandwich Principle” Immunochromatographic Assay

Test Procedure

1. Dispense 3 drops (100 µL) of specimen using the dropper into the circular sample well.
2. Interpret test results in 15 minutes.

Interpretation

1. Positive: Presence of coloured test and control band.
2. Negative: Only presence of coloured control band.
3. Invalid: Absence of coloured control band (Repeat with a new test device).

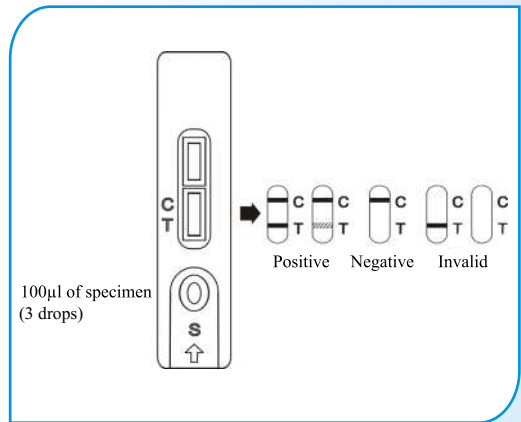

Product Code : FRERAIT10008

Order No. TBM/IMM/05 Rev. B

Imported and Marketed exclusively by :

ERBA diagnostics Mannheim GmbH

Mallastrasse 69-73, 68219 Mannheim / Germany
 Telephone: +49 621 8799770 Fax: +49 621 8799688
 Website: www.erbamannheim.com
 Email: sales@erbamannheim.com

Transasia Bio-Medicals Ltd.

Transasia House, 8 Chandivali Studio Road, Andheri (E), Mumbai - 400 072
 Tel.: (022) 40309000, Fax : (022) 2857 3030
 Email : transasia@transasia.co.in